

VINNO⁶

Ultrasound System Specifications

The premium performance of the full functional Portable VINNO 6 provides a fast and easy diagnosis by:

- Ultra-premium contrast and resolution imaging benefited from the first RF platform of the world
- All ranges of features, functions and probes
- Ease of use and ergonomic design

System Overview

Architecture

- The revolutionary RF platform, **The First In The World**, allows for more accurate information. This platform transfers all RF data for computing without any information loss. It has a much better advantage in detail imaging than current advanced platforms.
- Thanks to the RF platform, it allows the development of many RF-based processing algorithms, which have ultra-premium contrast and resolution imaging
- This unique platform is capable of processing multiple data streams simultaneously
- World-class Up to 23MHz imaging performance
- Directional-enhanced information compiling for more tissue detail and reduction of angle-generated artifacts
- Next generation adaptive image processing for noise and artifact reduction that improves tissue presentation and edge definition



- Fully independent, triplex multiple mode operation for easy in Doppler procedures
- Multi-processors allow simultaneous mode changes and support for advanced system functionality
- World-class design to be thinner and lighter
- First in class to introduce capacity touch panel in portable ultrasound system to simplify workflow and customized workflow possibility
- Less 20 sec boot up time for easy mobile ability

Applications

- Abdomen
- Obstetric
- Gynecology
- Cardiology
- Urology
- Vascular
- TCD
- Small Parts
- Pediatrics
- Intra-operative

Imaging features

- 2D grayscale imaging
- Harmonic imaging both in tissue harmonic and pulse inversion harmonic technologies
- VFusion, directional-enhanced information compounding
- Vspeckle , specialized and adaptive imaging processing to remove speckle noise artifacts and enhance tissue edge for clarity and accuracy
- VTissue, the advanced adaptive image processing to compensate for sound and speed variation in different tissue
- Auto imaging optimization
- Easy Comparative Function to compare previous exam
- Color Doppler imaging
- Power Doppler imaging
- Pulse wave Doppler imaging
- Simultaneous 2D and M mode
- Duplex 2D/PW Doppler
- Triplex 2D/Color/PW Doppler
- High PRF pulsed wave Doppler
- Continuous wave Doppler
- Zoom
- FULL screen imaging to enlarge imaging size
- Dual real time imaging without compromising imaging size
- PView for panoramic imaging (Optional)*
- TView for trapezoidal imaging
- Free 3D
- 3D imaging
- 3D/4D HQR (High Quality rendering) (Optional)*
- Spatio Temporal Image Correlation (STIC) (Optional)*

- Real-time grayscale 4D
- Tomographic display (MCUT)
- Three leads ECG function(option) *
- Tissue Doppler (TD) mode
- Tissue Velocity Imaging (TVI) mode*
- Tissue Velocity M mode (TVM)(option)*
- Multi-angle M mode (option)*
- Auto IMT function
- Auto NT*
- Smart 3D Volume Calculation(option) *
- Inversion mode(option)*
- Magic cut(option)*
- Smart touch panel 3D/4D operation(option)*
- Free view(option)*
- Elastic Imaging(EI) (option)*
- VGuide

Standard features

- Up to 25Mhz high frequency in system platform. Up to 23MHz's probes are supported
- RF platform and RF data processing
- Up to 1000 seconds cine storage
- 250GB SSD quick boot up and storage
- Patient information database
- Image archive on hard drive
- Quick store to USB memory stick
- Quick store to hard drive
- Report package
- Quick print to B/W and color thermal video printer
- Network storage and printing
- Full measurement and analysis package
- Real time auto wave Doppler track and calculations
- Vascular calculations
- Cardiac calculations
- OB calculations and tables

- Gynecological calculations
- Urological calculations
- Renal calculations
- Volume calculations
- Wireless networking for easy data sharing, storage and printing*(optional)
- Up-to-date connectivity and data management solutions, wireless , LAN, integrated database*(optional)
- Capability to send data to mobile by mail and blue tooth*(optional)
- Total mobile medical solution for remote data transfer and diagnostic
- DICOM compatibility*(optional)
- 2USB ports
- 6 TGC slides
- Average 4 multiple adjustable frequency in every probe and mode
- Up to 512 line density

Ergonomics

- Unique human oriented design for comfort and convenience
- 15.6-inch high resolution flat panel display with nearly infinite positioning adjustments
- Easy to carry by integrated handle
- Full integrated probe to reduce overall space
- Integrated touchable alphabetic keyboard
- Integrated capacity touch panel to easy and simplify workflow
- Cart support up to 15.60mm up/down operation panel*(optional)
- USB DVDRW *(optional)

Keyboard

- Highly sensitive 8 inch capacity touch panel
- Intuitive, configurable and touchable interactive operation interface
- Ergonomic hard keys for general ultrasound operations
- 6 TGC slides, functionality at any depth
- Backlight keys

Image display screen

- 15.6 inch high resolution IPS, LCD technology, pixel resolution
- Big angle tilting capability

Peripherals

- B&W thermal video printer: Sony UP-D897MD (optional)
- Color thermal video printer: Sony UP-D25MD (optional)
- Memory stick (optional)

Dimensions and Weight

- Length: 387mm
- Width: 340mm
- Depth: 72mm
- Weight: without accessories approx. 3.8kg

Electrical Power

- Voltage: 100-240V AC
- Frequency: 50/60Hz
- Power: Max.170VA

Transducers

Transducer Technology

- Xcen technology for wideband frequency
- Pure wave technology for high resolution imaging
- Unique and high technical Xcen probe connector to adapt all different type of VINNO product models

Transducer types

- Convex array
- Linear array
- Phase array
- 4D probe
- Endocavity probe
- Micro-convex array

Transducer selection

- Electronic switching of transducers
- User customizable imaging presets for each transducer and application
- Automatic dynamic receiving focus in all transducers
- Multiple adjustable transmit focal zone, up to 8 focal zoom

G2-5C Broadband Curved Array

- Field of view: 72 degree
- Convex radius: 50mm
- Application: abdomen, OB/Gyn, urology, pediatric
- Frequency range: 1.4 -5.6MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes
- Reusable biopsy guide available

D3-6C broadband curved array volume probe

- Field of view: 78 degree
- Convex radius: 40mm
- Application: abdomen, OB/Gyn, urology
- Frequency range: 1.9 - 7MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic, 3D/4D grayscale and 3D color modes
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

D3-6CX broadband curved array volume probe

- Field of view: 68 degree
- Convex radius: 40mm
- Application: abdomen, OB/Gyn, urology
- Frequency range: 1.9 - 7MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic, 3D/4D grayscale, 3D color
- Multi-imaging frequency setting in 2D, 3D/4D, Harmonic, color Doppler and Wave Doppler modes

F2-5CE Broadband Curved Array

- Field of view: 60 degree
- Convex radius: 60mm
- Application: abdomen, OB/Gyn, urology, pediatric
- Frequency range: 1.4 -5.6MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes
- Reusable biopsy guide available

G4-9M broadband micro convex array

- Field of view: 138 degree
- Convex radius: 12mm
- Application: pediatric, abdomen, cardiac
- Frequency range: 3 - 10MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

G4-9E broadband micro convex endocavity array

- Field of view: 180 degree
- Convex radius: 9.6mm
- Application: Ob/Gyn, urology
- Frequency range: 3 - 10MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes
- Reusable biopsy guide available

F4-9E broadband micro convex endocavity array

- Field of view: 15.60degree
- Convex radius: 10mm
- Application: Ob/Gyn, urology
- Frequency range: 3 - 10MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes
- Reusable biopsy guide available

D4-9E broadband micro convex 4D endocavity array

- Field of view: 125 degree
- Convex radius: 10mm
- Application: Ob/Gyn, urology
- Frequency range: 3 - 10MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic , 3D/4D grayscale, 3D color
- Multi-imaging frequency setting in 2D, 3D/4D, Harmonic, color Doppler and Wave Doppler modes

X4-12L broadband linear array

- Fine pitch, high resolution
- Applications: vascular, small parts
- Frequency range: 4.5 -13MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

F4-12L broadband linear array

- Fine pitch, high resolution
- Applications: vascular, small parts
- Frequency range: 4.5 -13MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

I7-18L broadband linear array(Hock stick)

- Fine pitch, high resolution
- Applications: vascular, small parts, Hock stick
- Frequency range: 6.5 -18MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

X6-16L broadband linear array

- Fine pitch, high resolution
- Applications: vascular, small parts
- Frequency range: 6.5 -18MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

X10-23L broadband linear array

- Fine pitch, high resolution
- Applications: small parts
- Frequency range: 10 -23MHz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

I4-11T broadband linear array


- Fine pitch, high resolution
- Frequency range: 4.2-11Mhz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

G1-4P phased array

- Applications: cardiac, abdomen, Ob/Gyn, Urology
- Frequency range: 1.35-4.3Mhz
- Pulsed wave Doppler, continuous wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

S1-6P phased array

- Single Crystal technology
- Applications: cardiac, abdomen, Ob/Gyn, Urology
- Frequency range: 1.9-7Mhz
- Pulsed wave Doppler, continuous wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

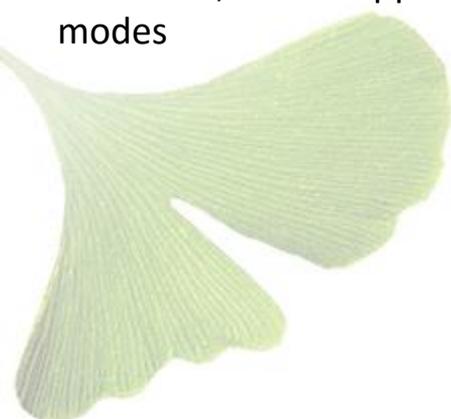

U5-15.6LE broadband linear array

- Fine pitch, high resolution
- Applications: small parts, specially for breast, vascular
- Frequency range: 5 -15.6Mhz
- Pulsed wave Doppler, color Doppler, power Doppler, harmonic
- Multi-imaging frequency setting in 2D, Harmonic, color Doppler and Wave Doppler modes

Advanced Imaging controls

VFusion

- Available on all transducers and for 2D, 3D/4D
- Up to 5 levels of directional imaging fusion to enrich information
- Operate in conjunction with VSpeckle, harmonic imaging

VSpeckle

- Available on all transducers and for 2D, 3D/4D
- Virtually eliminate speckle noise artifact and dynamically enhances tissue margins
- Selectable multiple levels of speckle noise reduction and smoothing
- Operates in conjunction with VFusion and harmonic imaging

VTissue

- Special imaging processing to adapt to the speed of the ultrasound variation in different tissue
- Improved conspicuity of lesions, such as stone and tendon

3D/4D

- 3D/4D rotation
- Grayscale imaging controls
- Selectable rendering approaches
- Unique high quality rendering algorithm
- Selectable gray maps
- Multi slide cutting
- Cineloop 3D
- Review volume



3D/4D HQR (High Quality Rendering) (Optional)*

- Amazing high image quality
- Extreme realistic rendering images
- Similar operation as normal rendering

Spatio-Temporal Image Correlation (STIC) (Optional)*

- Visualize the fetal heart or an artery
- One complete heart cycle represented
- Using 3D static acquisition

Tissue Doppler (TD)

- Present wall motion spectrum by using Doppler principle
- Provide wall motion direction and velocity information

Tissue Velocity Imaging (TVI)*

- Color codes the velocities in tissue
- Present tissue color imaging by using Doppler principle
- This color image is overlaid onto the 2D image
- Captures low flow but high amplitude signals associated with wall motion

Smart Touch 3D/4D Operation(Optional)

- Fully utilize touch panel possibility for easy operation, such as rotation 3D rendering image, move ROI, create line by finger

Free View(Optional)

- Provide any plane view to visualize the internal tissue information
- Improve the contrast resolution to facilitate the detection of diffuse lesions in organs

Inversion mode*(Optional)

- This render mode is used to display anechoic structures such as vessels
- It invert the gray values of the rendered image, such as black image information become white and vice versa

Magic Cut*(Optional)

- Ability to edit images, make possible to cut away structure obstructing the view in the ROI
- Several cutting methods available

Smart 3D Volume Measurement(Optional) *

- Trace the margin of the irregular circle in different slices of volume data in irregular shape
- Automatically report the volume of the irregular object

Auto Follicle(2D/3D)(Optional) *

- Just click on the area of follicle in B mode, the area of this follicle will be reported automatically
- Report the area of different follicle in the volume data automatically

Advanced Imaging controls (cont.)

Tissue Velocity M mode(TVM) *

- Color codes the velocities in tissue
- Present wall motion spectrum based on tissue moving
- This color image is overlaid onto the 2D image
- Captures low flow but high amplitude signals associated with wall motion

Multi-angle M mode *

- Sample on moving tissue from multi-angle
- Present wall motion spectrum based on tissue moving

Auto NT (Nuchal Translucency) measurement*

- Automatically detect Nuchal Translucency in interest box
- Automatically report thickness result of NT

Auto IMT (Intima-Media Thickness) measurement

- Automatically detect intima media thickness in interest box
- Automatically report the result of IMT
- Available in linear probe

Elastic Imaging(EI) *

- use the probe to press tissue artificially
- color codes the elasticity in tissue
- This color image is overlaid onto the 2D image

Next generation RF-based image processing

- Available on all imaging transducers in 2D grayscale modes
- Virtually eliminates speckle noise artifact and dynamically enhance tissue edge
- Operates with other real-time processing algorithms

Imaging modes

2D Imaging

- Pre-defined ATGC (adaptive temporal gain compensation) curves optimized for consistently excellent imaging
- B/M acoustic output: 0-100%
- Depth: able to adjust from 1 to 36cm
- Select between 1 to 8 transmit focal zones
- Reverse function: on/off
- VFusion function
- VSpeckle function
- Harmonic imaging both tissue harmonic and phase inversion
- Cineloop image review
- Selectable 2D line density
- Dual imaging with independent cineloop
- 256(8 bit) gray level
- Up to 8 focus zone adjustable
- Multiple color maps with chroma imaging
- FULL screen imaging to larger image size
- Multi frequency: probe dependent
- Gray filter: 7 steps
- Persistence: 8steps

- Selectable image angles, probe dependent
- Gain: 0-100%
- Dynamic range: 30-280 db
- VSharpen to enhance edge contrast
- Smooth to improve spatial resolution

Harmonic Imaging

- Supports both tissue harmonic and phase inversion imaging (transducer and frequency dependence)
- Second harmonic processing to reduce artifacts and improve image clarity
- Maximize detail resolution and enhance contrast
- Available on all imaging transducers
- Extends high performance imaging capabilities to all patient body types

M mode

- Selectable sweeping rates
- Time marks: 0.025 – 0.5 second
- Selectable display format prospective or retrospective (1/3-2/3, 1/2-1/2, 2/3-1/3, side by side 1/2-1/2, side by side 1/3-2/3, full screen)
- Chroma colorization with multiple color maps
- Cineloop review for retrospective analysis of M-mode data
- 256 gray levels

Color Doppler mode

- Available on all imaging transducers
- Automatically adapts transmit and receive bandwidth processing based on the color box position

- Cineloop review with full playback control
- Steering on linear array transducers
- Selectable in baseline, line density, flash reduction, persistence, maps, frequency, PRF, wall filter, packet size, color level, sensitivity, focus position, acoustic power, and smooth
- Color gain
- Region of interest
- Baseline invert
- Simultaneous mode during PW mode
- Smoothing
- Wall filter
- Zoom

Power Doppler mode

- High sensitive mode for small vessel visualization
- Available on all transducers
- Cineloop review
- Multiple color maps
- Individual controls for gain
- Selectable baseline, line density, flash reduction, persistence, maps, frequency, PRF, wall filter, packet size, color level, sensitivity, focus position, acoustic power, and smooth
- Adjustable region of interest

Pulsed Wave (PW) Doppler

- Ultra high resolution spectral FFT rate
- Angle correction with automatic velocity scale adjustment
- Normal, invert display around horizontal zero line
- Selectable gray filter, dynamic range, frequency, PRF, wall filter, baseline, angle correct, sample volume
- Selectable sweep speeds: 8 steps
- Maximum velocity range: 12m/s

- PW acoustic output: 0-100%
- Selectable low frequency signal filtering with adjustable wall filter settings
- Selectable grayscale curve for optimal display
- Selectable chroma colorization maps
- Selectable display format prospective or retrospective (1/3-2/3, 1/2-1/2, 2/3-1/3, side by side 1/2-1/2, side by side 1/3-2/3, full screen)
- Auto function to optimize spectral Doppler displ.
- Digitally enhanced stereo output
- 256 gray levels
- Post-processing in frozen mode includes map, baseline, invert and chroma
- Simultaneous or duplex mode of operation
- Simultaneous 2D, color Doppler, pulsed Doppler
- High PRF capability in all modes including duplex and triplex

Continuous Wave Doppler (CWD)

- Cardiac sector array transducer only
- Maximum velocity range: 19m/sec

Pview*

- Real time extended field of view composite imaging
- Ability to back up and realign the image during acquisition
- Full zoom, cineloop review and image rotation capabilities
- User can measure distance and area
- Measurement can be made on individual frames during cineloop review

- Available on linear transducers

Tview

- Expand view of scanning
- Available on linear transducers

Auto

- Intelligent one button automatic optimization in 2D and Doppler modes
- Automatically adjust PRF and baseline in Doppler

Tissue Doppler Imaging (TD) *

- Present wall motion spectrum by using Doppler principle
- Provide wall motion direction and velocity information
- Available on all sector transducer for cardiac imaging
- Gain

Tissue Velocity Imaging (TVI) *

- Color codes the velocities in tissue
- Present tissue color imaging by using Doppler principle
- This color image is overlaid onto the 2D image
- Captures low flow but high amplitude signals associated with wall motion
- Available on all sector transducer for cardiac imaging
- Tissue velocity M mode display for wall motion(*optional*)
- Gain
- Velocity

Touch Panel Interface

2D mode

- New patient
- BodyPattern
- Archive
- Probe&APP
- Comments
- End exam
- Sys setting
- Report
- PView
- Fullscreen
- L/R
- U/D
- Center line
- VTissue
- VSpeckle
- VFusion
- Gray Filter
- Persistence
- Display Format
- Image reference
- Maps
- Frequency
- Focus position
- Focus #
- Dynamic Range
- Line density
- VSharpen
- Biopsy
- Image angle
- Focus width
- Smooth
- Acoustic power

M Mode

- New patient
- BodyPattern
- Archive
- Probe&APP
- Comments
- End exam
- Sys setting
- Report
- L/R format
- U/D format
- Maps
- Dynamic range
- Acoustic power
- Sweep speed
- Gray filter
- VSharpen
- ECG

CF mode

- New patient
- BodyPattern
- Archive
- Probe&APP
- Comments
- End exam
- Sys setting
- Report
- Invert
- Full Screen
- L/R
- U/D
- Baseline
- Flash Reduction
- Line density
- Persistence
- Display format

- Image reference
- Maps
- Frequency
- PRF
- Wall filter
- Packet size
- Colorlevel
- Sensitivity
- Focus position
- Acoustic power
- Smooth

PW/CW mode

- New patient
- BodyPattern
- Archive
- Probe&APP
- Comments
- End exam
- Sys setting
- Report
- Invert
- Triplex
- Display format
- Sweep speed
- Gray filter
- Dynamic range
- Trace sensitive
- Auto trace
- Mode/direction
- Maps
- Frequency
- PRF
- Wall filter
- Baseline
- Angle correct
- Sample volume
- Audio Volume
- Spectrum optimize

- Acoustic power

3D mode

- Comments
- BodyPattern
- Back to B
- Start3D
- ROI shape
- Render
- Display format
- Image reference
- View
- Gray map
- VSpeckle
- Quality
- Threshold
- Transparency
- Volume angle
- Movement step (after data acquisition)
- HQ Light(after data acquisition)
- Rotation angle (after data acquisition)
- Rotation direction (after data acquisition)

4D mode

- Comments
- Body Pattern
- Back to B
- Start 4D
- ROI shape
- Movement step
- Rotation direction
- Render
- Display format
- Image reference
- View
- Gray map
- VSpeckle

- Quality
- Threshold
- Transparency
- Volume angle

System Feature

Display modes

- Simultaneous capability
 - 2D/PW/CW
 - 2D/CF or PDI
 - 2D/M
 - Dual, 2D/2D
 - Dual, 2D/2D+CF or PDI
 - Dual, duplex and triplex
 - Duplex and Triplex mode
 - Quad display in 3D/4D application
 - 9 slice images display in 3D/4D

application

- Time line display
 - Independent dual 2D/PW or CW
 - Timed based sweep update mode

Display annotation

- Institution/hospital name
- Date: 2 types selectable, YY/MM/DD, MM/DD/YY
- Time: 2 types selectable, 24hours and 12 hours
- Operator identification
- Patient name, first, last
- Patient identification: 30 characters
- Gestational age from LMP/EDC/GA/BBT
- VINNO image symbol: Ginkgo leaf
- Power output index
 - MI: mechanical index
 - TIS: thermal index soft tissue
 - TIC: thermal index cranial (Bone)

- TIB: thermal index bone
- Probe orientation marker: coincide with a probe orientation marking on the probe
- Gray/color bar
- Measurement result window
- Probe type
- Application name
- Image depth
- Imaging parameters by mode
 - 2D/M mode: acoustic power output, gain, frequency, frame rate, dynamic range
 - Color mode: color acoustic power output, color gain, color flow frequency, PRF, wall filter
 - PW/CW mode: Doppler acoustic power output, Doppler gain, Doppler frequency, PRF, wall filter, sample depth
- Focus zone marker
- Body pattern
- PW and CW scale markers: time/speed
- M scale markers: time/depth, time
- System measurement display
- System message display
- Biopsy guide line
- Heart rate

Cineloop

- Acquisition, storage in memory and display of up to 1000 seconds long of 2D, color and PW/CW images for review

Compare

- Compare live imaging with stored imaging.

Quick save feature

- The system provides quick save function through USB stick, internal/external HDD during or after exam
- Configurable saving file format, VRD (VINNO Raw Data), DICOM, JPEG, BMP, PNG and AVI

Physio

- One 3-lead ECG input*(Optional)
- Gain, sweep rate and display position controls
- Automatic heart rate calculation and display
- Fault condition display

Archive

- Patient data input which include patient ID, name, nationality, birth date, sex, exam physician, quality check, exam operator
- Physical data such as weight, height
- Patient exam management
- Patient exam images storage and management
- Import VRD format data into the system from outside media, such as USB stick, external HDD
- Export patient data into outside medias

Report

- Automatically pull patient data into the report
- Automatically load measurement worksheet into the report
- Pull related exams' images into the report
- Write comments in the report
- Print report through network or local printer

Connectivity

- Standard connectivity features
 - Local print to on-board or off-board video printers through USB port
 - Page report print
 - Image export to removable media (external HDD, USB stick)
- Network linkage
 - Image export to network storage servers
 - DICOM export and retrieve *(Optional)
- Mobile data transfer solution by
 - Blue tooth*(Optional)
 - Email*(Optional)
 - Hot point connection
- VCloud * (Optional)
- DICOM workstation for remote diagnostic solution *(Optional)
- DICOM, JPEG, BMP, PNG, AVI
 - VRD and DICOM images stored in disc can be recalled on the VINNO system
 - JPEG, BMP, PNG and AVI images can be played on normal computers
- On-board patient exam storage
 - Direct digital storage of static image or cineloop images to internal hard disk drives
- Fully integrated user interface

Probes/application

- Selectable multiple applications
- Edit exist application preset
- Edit user defined preset
- Rename preset
- Return to factory preset
- Quick save user defined parameters in related application

Safety Conformance

- Regulatory Notice:

This device is tested to meet all applicable requirements in relevant. According to 93/42 EEC, it is class IIa medical device.

- Conformity to Standards:

- IEC 60601-1 : 2012 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance
- IEC 60601-1-2:2007 Electromagnetic compatibility - Requirements and tests
- IEC 60601-1-6:2010 Usability
- IEC 60601-2-37:2007 Medical electrical equipment - Particular requirements for the safety of ultrasonic medical diagnostic and monitoring equipment
- IEC 61157:2007 Declaration of acoustic output parameters
- ISO 10993-1:2009 Biological evaluation of medical devices
- IEC 62304:2006 Medical device software –Software life cycle processes
- IEC 62366:2007 Medical devices - Application of usability engineering to medical devices
- Council Directive 93/42/EEC on Medical Device
- WEEE according to 2012/19/EU
- RoHS according to 2011/65/EU

Measurement and Analysis

Generic Measurement in 2D mode

- Depth
- Distance
- Perimeter
 - Length and width method
 - Ellipse method
 - Polygon method
 - Spline method
 - Tracing method
- Area
 - Length and width method
 - Ellipse method
 - Polygon method
 - Spline method
 - Tracing method
- Volume
 - Single line method
 - Dual line method
 - Triple line method
 - Single ellipse method
 - Single ellipse and single line method
- Angle
- Stenosis
 - Diameter method
 - Square meter method
- A and B ratio
 - Diameter ratio
 - Square meter ratio

Generic Measurement in CFM mode

- CFV
 - point
 - profile



Generic Measurement in M mode

- Depth
- Distance
- Time
- Slope
- Heart rate
- Stenosis
- A and B ratio
 - Diameter ratio
 - Time ratio
 - Velocity ratio

Generic Measurement in PW mode

- Speed (include PV (Peak Velocity))
- Time (include AT (Accelerate Time))
- Acceleration
- PS (Peak Speed in systole period)
- ED (The speed in the end of diastole period)
- MD (Minimum speed in diastole period)
- TAMAX (maximum speed in time average)
- TAMEAN (mean speed in time average)
- TAMIN (minimum speed in time average)
- PI (Pulsatility Index)
- RI (Resistance Index)
- PS and ED ratio
- ED and PS ratio
- A and B ratio (A/B ratio)
 - Speed ratio
 - Time ratio
 - Acceleration ratio
- FLOWVOL (Flow Volume)
- MaxPG (maximum pressure gradient)
- MeanPG (Mean pressure gradient)
- SV (Stroke Volume)
 - Each volume diameter cardiac

- Time mean speed in each stroke volume
 - Cardiac output
- Heart rate

Abdominal Measurement

- General abdomen
- Difficult abdomen
- Kidney
- Renal vessel
- Abdominal trauma

Small Part Measurement

- Thyroid
- Breast
- Testis
- Musculoskeletal
- Upper and lower extremity joint
- Nerve block

Vessel Measurement

- Carotid artery
- Upper artery
- Upper vein
- Lower artery
- Lower vein
- Vessel puncture
- Transcranial Doppler

Gynecology Measurement

- Uterus and Pelvis
- Follicle

Urology Measurement

- Bladder
- Prostate
- Renal
- Kidney and ureter
- Pelvic Floor dysfunction

Pediatric Measurement


- Neonatal Head
- Neonatal Abdomen
- Pediatric Abdomen
- Pediatric Hip
- FAST

Obstetrics Measurement

- OB Early
- OB Mid
- OB Late
- Fetal Heart

Cardiac Measurement

- General
- LV
- MV
- Ao
- AV
- LA
- RV
- TV
- PV
- RA
- System

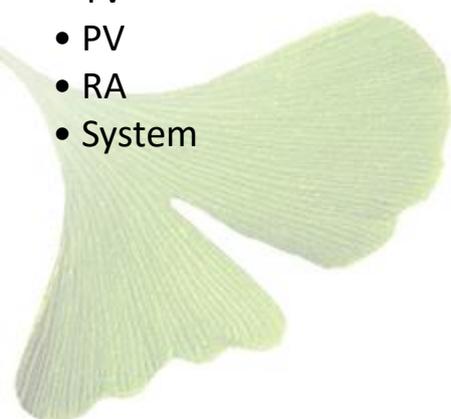

VINNO Technology (Suzhou) Co., Ltd

VINNO is focusing on producing premium diagnostic ultrasound development to provide customer clinical value through *Continuous Innovation, Excellent Performance and Accessible Solutions.*

Thanks you for your interest in VINNO. You can contact us by phone and email (below) or contact our local representatives.

VINNO Technology (Suzhou) Co., Ltd

5F,A Building,NO.27 Xinfu Rd,Suzhou Industrial Park,215123, China(215.6123)

Tel: +86 512 62873806

Fax: +86 512 62873801

email address: vinno@vinno.com

website: www.vinno.com